cefar[®]

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GEBRAUCHSANWEISUNG | OPERATING INSTRUCTIONS | MANUAL DE INSTRUCCIONES

MODE D'EMPLOI | ISTRUZIONI PER L'USO | GEBRUIKERSHANDLEIDING

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OPERATING INSTRUCTIONS – ENGLISH

Equipment and accessories	16
CEFAR PRIMO	16
CEFAR PRIMO EASY	16
Information concerning both CEFAR PRIMO and CEFAR PRIMO EASY	17
Practical use CEFAR PRIMO	18
Practical use CEFAR PRIMO EASY	20
Replacement of batteries	21
Care instructions	22
Precautionary measures	22
Troubleshooting	23
Technical data	24
Key to the symbols	24

EQUIPMENT AND ACCESSORIES

CEFAR PRIMO

CEFAR PRIMO is a two-channel stimulator (four electrodes) intended for TENS treatment – transcutaneous electrical nerve stimulation. The stimulator has independent channels, i.e., two different forms of stimulation may be used simultaneously. You can choose between 9 pre-set programs. Besides conventional (P1), burst (P2) and modulated pulse duration stimulation (P3), CEFAR PRIMO offers mixed frequency stimulation, 2 Hz/15 Hz (P4) and 2 Hz/80 Hz (P5). With mixed frequency stimulation activation of both coarse and muscle based nerve fibres is possible, offering the patient potentially a more effective treatment. With mixed frequency stimulation one amplitude for each respective frequency is set to make the treatment comfortable. Program 6

(10 Hz) is a program optimized for treating nausea. Program 7 is a conventional stimulation with narrow pulse duration (60 μ s), which is suitable for facial stimulation. Program 8 and 9 have a pre-set timer.

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CEFAR PRIMO EASY is a two-channel stimulator (four electrodes) with non-independent channels. You can choose between 3 pre-set programs; conventional (P1), burst (P2) and modulated pulse duration stimulation (P3). A timer, connected to both channels, can be activated. If the timer is activated the stimulator automatically switches off after 30 minutes.

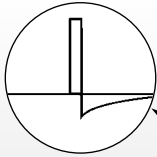
INFORMATION CONCERNING BOTH CEFAR PRIMO AND CEFAR PRIMO EASY

Precise amplitude setting allows exact adjustment of current in steps of 0.5 mA. A key lock which prevents unintentional changes in settings, ensures safe treatment.

The stimulator runs on two 1.5 V AA batteries. A battery indicator shows when the batteries need to be replaced. If not used, the stimulator automatically switches off after 5 minutes.

When the stimulator is not in use the cables may conveniently be stored directly on the unit with a belt clip to hold them in place (see picture p. 97).

The stimulator should be used with skin electrodes for nerve and muscle stimulation. These are either conductive carbon rubber (used with gel and tape or adhesive gel) or self-adhesive and come in various sizes. Their conductive properties deteriorate gradually and the carbon rubber electrodes should be changed after 4–6 months of use. The service life of the self-adhesive multi-use electrodes is strongly dependent upon their care and they normally last about one month. For instructions on use see electrode and gel packages.

Wave form:

Asymmetrical biphasic pulse,
100% compensated.

Conventional TENS (single pulses).

Burst TENS (pulse train).

Modulated pulse duration TENS (single pulses
with modulated pulse duration).

3 sec. 3 sec.

Mixed frequency TENS (2 Hz in 3 sec./15 Hz or 80 Hz
in 3 sec.). Only available in CEFAR PRIMO.

PRACTICAL USE CEFAR PRIMO

Turns the stimulator on and off.
The stimulator can be switched off even when the key lock is activated.

Selection between programs 1–9 for the left channel.
Changing of programs can only be done when both channels are set to 0.0 mA.

Selection between programs 1–9 for the right channel.
Changing of programs can only be done when both channels are set to 0.0 mA.

Amplitude increase for each channel.
The button can be held in for a continuous increase of amplitude.
NOTE! Increase the current strength cautiously.

Amplitude decrease for each channel.
The button can be held in for a continuous reduction of amplitude.
Deactivation of key lock.

When using a program that has an integrated timer (P8 and P9) a clock symbol is shown.

A key symbol is shown when the key lock is activated.

1. Attach the electrodes to the cables. If carbon rubber electrodes are being used, spread on an even layer of gel. Wash and dry the skin well before attaching the electrodes.
2. Turn the stimulator on with . If you wish to use the same settings as last time you stimulated, you can start stimulation directly by setting the desired amplitude with  for each channel.
3. To choose program press  for each channel until the wanted program is shown. **NOTE!** This can only be done when both channels are set to 0.0 mA.
4. Start stimulation by selecting the desired amplitude with  for each channel.

NOTE!

20 seconds after the amplitude has been set the key lock is automatically activated. Deactivate the key lock by pressing one of the  buttons.

Note with mixed frequency TENS (P4 and P5):

One amplitude for each frequency is being set, to make the stimulation comfortable. Instructions: First set one amplitude for conventional stimulation (15 Hz or 80 Hz) and then, when the stimulator after 3 seconds switches over to burst stimulation (2 Hz), set the amplitude for this frequency. mA is lit with conventional stimulation and blinking with burst stimulation.

Program:	Stimulation form:	Frequency:	Pulse duration:	Timer:
P1	Conventional TENS	80 Hz	180 µs	---
P2	Burst TENS	2 Hz	180 µs	---
P3	Modulated pulse duration TENS	80 Hz	70–180 µs	---
P4	Mixed frequency TENS	15 Hz/2 Hz	180 µs	---
P5	Mixed frequency TENS	80 Hz/2 Hz	180 µs	---
P6	Nausea treatment	10 Hz	180 µs	---
P7	Facial treatment	80 Hz	60 µs	---
P8	Conventional TENS	80 Hz	180 µs	30 minutes
P9	Burst TENS	2 Hz	180 µs	30 minutes

PRACTICAL USE CEFAR PRIMO EASY

Turns the stimulator on and off.
The stimulator can be switched off even when the key lock is activated.

Selection between programs 1–3 for both channels.
Changing of programs can only be done when both channels are set to 0.0 mA.

Amplitude increase for each channel.
The button can be held in for a continuous increase of amplitude.
NOTE! Increase the current strength cautiously.

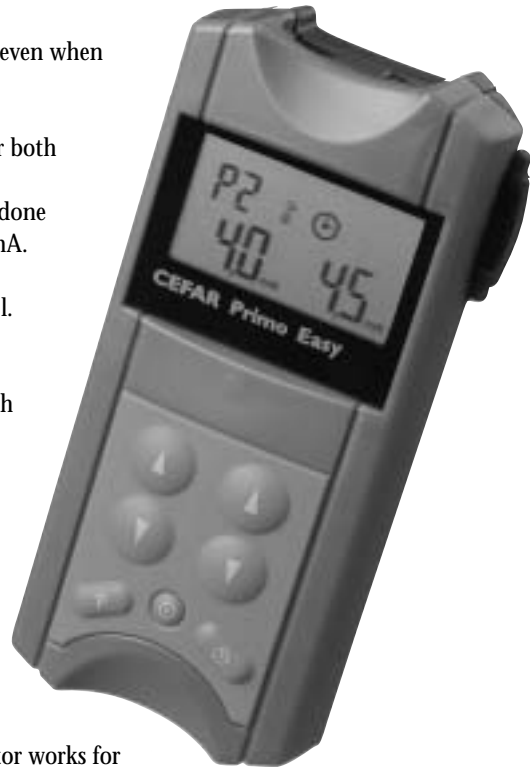
Amplitude decrease for each channel.
The button can be held in for a continuous reduction of amplitude.
Deactivation of key lock.

The timer works for both channels.
If the timer is activated the stimulator works for 30 minutes and is then automatically switched off. A clock symbol is shown when the timer is activated.

A key symbol is shown when the key lock is activated.

1. Attach the electrodes to the cables. If carbon rubber electrodes are being used, spread on an even layer of gel. Wash and dry the skin well before attaching the electrodes.
2. Turn the stimulator on with . If you wish to use the same settings as last time you stimulated, you can start stimulation directly by setting the desired amplitude with  for each channel.
3. To choose programs press  until the wanted program is shown.

NOTE! This can only be done when both channels are set to 0.0 mA.

4. Start stimulation by selecting the desired amplitude with  for each channel.

NOTE!

20 seconds after the amplitude has been set the key lock is automatically activated. Deactivate the key lock by pressing one of the  buttons.

Program:	Stimulation form:	Frequency:	Pulse duration:
P1	Conventional TENS	80 Hz	180 µs
P2	Burst TENS	2 Hz	180 µs
P3	Modulated pulse duration TENS	80 Hz	70–180 µs

REPLACEMENT OF BATTERIES

When? A battery indicator is lit on the display when the batteries are almost empty. As long as stimulation feels the same as usual you can continue the treatment. It is time to replace the batteries when stimulation starts diminishing or the stimulator turns off.

NOTE! Non-rechargeable batteries may not be charged due to the risk of explosion.

How? Turn off the stimulator. Remove the battery cover by sliding it downwards and then lifting it upwards and take out the batteries. Make sure the new batteries are placed correctly with respect to the polarity (note marks for + and -). See p. 97.

NOTE! Do not throw the batteries out, but recycle appropriately.

CARE INSTRUCTIONS

Taking care of and cleaning the Cefar equipment is simple with the following instructions:

1. Keep stimulator and accessories in the original case when they are not in use. It may, however, be practical to allow the electrodes to remain on the body between treatments. Carbon rubber electrodes can generally remain for 2–3 hours without the electrode gel drying out (does not apply to adhesive gel). They must then be taken off, washed and dried before being applied again. This is especially important for persons with sensitive skin. In connection with stimulation, make sure that the electrodes are firmly in place.
2. The carbon rubber electrodes and the skin are rinsed with water after use. Do not use detergent for the electrodes.
3. Self-adhesive multi-use electrodes are re-moistened if necessary with a few drops of water and kept air-tight (in a plastic bag) on protective paper when they are not in use.
4. Do not expose the stimulator to water. Dry with a damp cloth if necessary.
5. Do not jerk cords or connections.
6. The leads are best preserved if left attached to the stimulator between sessions.

PRECAUTIONARY MEASURES

1. **Warning!** Patients with pacemakers must not be treated with TENS.
2. Stimulate with precaution while treating angina pectoris.
3. Do not stimulate near the glomus caroticum or near the carotid arteries on the neck since a drop in blood pressure can occur.
4. Pregnant women should not be treated with TENS during the first trimester and should seek medical advice prior to using TENS.
5. Inspect the equipment prior to use.
6. The stimulator should be kept out of reach of children.
7. The treatment is a personalized medical prescription; therefore, do not lend the stimulator to anyone else.
8. The electrodes are only to be placed on healthy skin. Avoid skin irritation by ensuring that good contact is achieved between electrodes and skin: When using carbon rubber electrodes use plenty of electrode gel and avoid drying out by applying tape around all the edges of the electrodes (does not apply to the adhesive gel).
9. If skin irritation should occur, treatment should be temporarily discontinued. If problems continue, contact the health care provider who ordered treatment. Hypersensitivity to tape and gel can occur in isolated cases. The problem usually disappears when the tape or gel is changed to another type.
10. Do not use electrodes with a surface of less than 5 cm², as there may be

risk for suffering a burn injury. Caution should always be exercised with current densities $> 2 \text{ mA/cm}^2$.

11. Switch off stimulation before removing the electrodes from the skin. If an electrode comes off, shut off the stimulation before picking it up. Getting electrical stimulation through the fingers is unpleasant but not harmful.
12. Training aches can occur after stimulation but usually disappear within a week.
13. Never open the battery lid during stimulation.
14. If the stimulator is not used for some time (approximately 3 months), the non-rechargeable batteries should be removed from the stimulator.
15. Do not use the stimulator in the

vicinity of short-wave or microwave equipment (mobile telephones, for example), since this may affect the output power.

16. Stimulation should not take place while the patient is connected to high frequency surgical equipment. It may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
17. Only Cefar accessories may be used with the stimulator.
18. Use the stimulator only as stated in the operating instructions.
19. The stimulator meets the requirements of MDD 93/42/EEC, including the EMC requirements. This, however, is no guarantee that the equipment will not be affected by electromagnetic disturbances.

TROUBLESHOOTING

Stimulation does not feel the same as usual.

Check that all settings are correct (see Practical use p. 18-20) and make sure that the electrodes are right in place.

Stimulation current feels unpleasant.

1. Skin is irritated. Advice on skin care, see Precautionary measures p. 22.
2. Electrodes are too old and need replacement.
3. Insufficient electrode gel on the carbon rubber electrodes.

Current pulses are weak, or missing.

1. Do the batteries need replacement? See Replacement of batteries p. 21.
2. If the circuit is not closed, due to a cable breakage for example, the

amplitude for this channel is automatically turned off and 0.0 mA starts blinking. Cable breakage can be checked by pressing the cable's pins against one another while increasing the amplitude for this channel to about 10 mA. If the amplitude now turns off and 0.0 mA blinks the cable needs to be replaced.

3. A too high resistance is indicated by the amplitude dropping to 0.0 mA and 0.0 mA starts blinking.
4. Electrodes are too old and need replacement.

Service of the stimulator should be performed by Cefar or Cefar's distributors and should be carried out once per year. Cefar will only be responsible for service and repairs performed by Cefar or a distributor appointed by Cefar.

TECHNICAL DATA

CEFAR PRIMO is a nerve stimulator (TENS) with two independent channels and 9 pre-set treatment programs. The stimulator has buttons for setting of program and amplitude for each channel. An automatic key lock is activated 20 sec. after the amplitude has been set.

CEFAR PRIMO EASY is a nerve stimulator (TENS) with two non-independent channels and 3 pre-set treatment programs. The stimulator has buttons for setting of program, amplitude and timer. An automatic key lock is

activated 20 sec. after the amplitude has been set.

The stimulators run on either two non-rechargeable 1.5 V AA batteries or two rechargeable batteries, recharged in a separate charger. Treatment with electrical stimulation requires the stimulation current to penetrate the resistance of the skin and the electrode, about 1500 ohms. CEFAR PRIMO and CEFAR PRIMO EASY can penetrate this resistance and maintain a current of up to 60 mA. With a change in load from 100 to 1500 ohms, the stimulation current changes less than 5% from the set value.

	CEFAR PRIMO	CEFAR PRIMO EASY
No. of channels	2 (independent)	2 (non-independent)
Constant current ohm	Up to a resistance of 1500 ohm	Up to a resistance of 1500
Stimulation current/channel	0-60 mA	0-60 mA
Wave form	Asymmetrical biphasic pulse, 100% compensated	
No. of pre-set programs	9	3
Stimulation forms	Conventional Burst Modulated pulse duration Mixed frequency	Conventional Burst Modulated pulse duration - - -
Pulse duration	See program p. 19.	See program p. 21.
Frequency	See program p. 19.	See program p. 21.
Timer	See program p. 19.	30 minutes/OFF
Environment for storage, use and shipping	Temperature 10° C–40° C • Air humidity 30%–75% • Air pressure 700hPa–1060 hPa	
Power source	2 pcs 1.5 V AA non-rechargeable or 2 pcs 1.2 V AA rechargeable batteries	
Current consumption for one channel, 80 Hz, 30 mA	100 mA	100 mA
I r.m.s. max/channel	10 mA	10 mA
Size	120 x 53 x 33 mm	120 x 53 x 33 mm
Weight	c. 185 g	c. 185 g

Key to the symbols

Read the operating instructions prior to use.  Patient part type BF – Body Floating.
Complies with MDD 93/42/EEC requirements. Notified body SEMKO AB.
A reference library is available on Cefar's home page, www.cefar.se.

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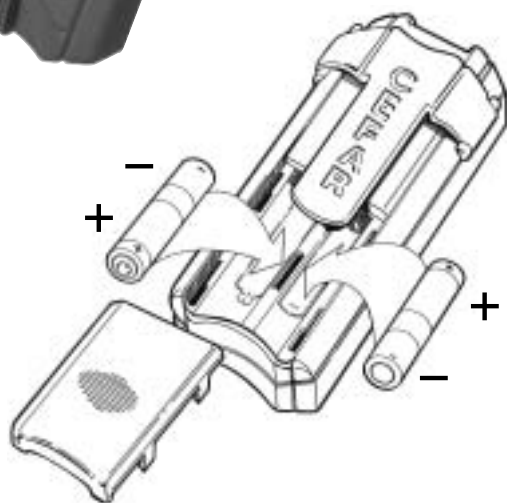
TENS | NMES REHAB | NMES SPORT IONTOPHORESIS | ELECTROACUPUNCTURE

ABOUT CEFAR

CEFAR Medical has conducted research and product development in electrical nerve and muscle stimulation since its establishment in 1975, and is globally represented through a network of subsidiaries and distributors.

Cefar's stimulators are used for TENS, NMES, electroacupuncture and iontophoresis and send electrical impulses via electrodes placed on a chosen body part. Electrical nerve and muscle stimulation has no side effects.

Business areas

- Medical - Professional healthcare - (pain relief, wound healing, nausea treatment, muscle and movement rehabilitation, non-invasive transdermal drug delivery and incontinence treatment)
- Sport/Fitness/Well-being - Consumer healthcare - (warm-up, muscle training, muscle relaxation, recovery, rehabilitation, pain relief, aesthetics, toning and massage)
- Pain Assessment (pain assessment and pain analysis)

References

The areas of application for electrical nerve and muscle stimulation within the medical field are developed through clinical research. Electrical nerve and muscle stimulation as an active method for training and recovery is further developed through a close cooperation with medical experts and top athletes.

cefar[®]

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Täyttää MDD 93/42/EEC -vaatimukset. SEMKOn AB hyväksymä.